## Bluetooth Antenna Configuration

help.grandcare.com  $\rightarrow$  technical help  $\rightarrow$  bluetooth antenna configuration

## Overview



The wireless health devices used with the systems all communicate via Bluetooth. So any system that includes health device peripherals (*blood pressure monitor, glucometer, pulse oximeter, or weight scale*) must have a Bluetooth antenna.

## **Supported Model**

All systems require an external Bluetooth antenna. Many different Bluetooth antennas will work with the system, but not all. See the Supported Hardware List for the current selection of supported Bluetooth antennas.

## **Installing a Bluetooth Antenna**

To install and configure a Bluetooth antenna on a system:

- 1. At the system, plug the antenna into any available USB port.
- 2. Press the "Settings" button
- 3. Enter the 4-digit System PIN
- 4. Press the "Peripherals" button
- 5. Press the "Restart Drivers" button
- 6. After the drivers have restarted, and the Peripherals screen displays again, verify that the Bluetooth row is green.



Note - it is not necessary to select the port or to turn on the Bluetooth antenna.

00-01-OUM-0026 Rev C (copy)