

# Quick Start Setup

---

## Set up your touchscreen and power it on in **Four Easy Steps**

### STEP 1

---

#### Place the System

Place the GrandCare System in a location convenient to the resident.

### STEP 2

---

#### Plug in the Adapter

Follow the provided manufacturer's instructions to assemble the AC/DC adapter and the AC power cord. Plug the power cord into an electrical outlet.



### STEP 3

---

#### Turn it On

Press and release the "[Power Button](#)" on the touchscreen.

The system takes a few minutes to completely load and display the screen.

**This system should never be turned OFF. It is designed to run all the time, collecting sensor information, updating tables and graphs, and sending designated alerts. These things can only happen while the system is running.**


## STEP 4

### Setup Wizard

After starting up the system, the [Setup Wizard](#) will automatically guide you through connecting to the internet and adding health devices.

If the Wizard does not start:

1. Touch the [Settings Button](#) in the lower-left corner of the screen.
2. Enter the "[System PIN](#)": 1111 and select "**System**".
3. Touch "**Setup Wizard**," then "**Yes**".
4. Select "**Restart Touchscreen**".

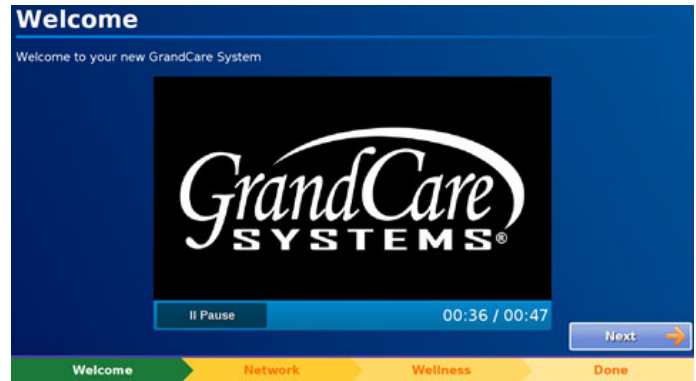

### Additional Information

[Safety Precautions](#)

[Cleaning Instructions](#)

[Packaging and Disposal](#)