

Power Button

help.grandcare.com → power button

Important

This system should *never* be turned OFF. It is designed to run all the time, collecting sensor information, updating tables and graphs, and sending designated alerts. These things can only happen while the system is running.

To Turn the System ON

- Press the power button one time. (Only press it once.)
- The system takes a few minutes to completely load and display the screen.

To Turn the System OFF



Never unplug the system while it's running.

In the rare situation that the system must be turned OFF, follow these instructions:

- DO NOT unplug the system to turn it off.
- DO press and release the power button. (Only press it once.) The system will go through the shut-down process, and then shut off.

Where is the Power Button Located?



Power Button is on the *front* of Asus computers



Power Button is on the *side* of MSI computers

GC-01-OUM-0051 Rev A (copy)