

Bluetooth Antenna

All [Wellness Devices](#) communicate with the systems via Bluetooth. Systems that include health devices must have a Bluetooth Antenna.

Installing a Bluetooth Antenna

To install and configure a Bluetooth antenna on a system:

Through the Touchscreen Setup Wizard

The first time you turn on a new GrandCare System, the [Setup Wizard](#) will start automatically and guide you through setting up your [Wellness Devices](#), including the Bluetooth antenna.

This is the easiest way to add devices, as the system will guide you through the setup process step-by-step.

If you are adding a device to an already configured system, see the [Setup Wizard](#) page for instructions on re-starting the Wizard.

Manually Through the Touchscreen

1. At the system, plug the antenna into any open **USB port** on the back of the touchscreen.
2. At the **Touchscreen**, touch the [Settings Button](#).
3. Enter the [System PIN](#).
4. Select the "**Peripherals**" button.
5. Touch "**Restart Drivers**".
6. After the drivers have restarted, the screen displays again. **Verify that the Bluetooth row is green.**



Supported Models

All systems require an external Bluetooth antenna. Many Bluetooth antennas will work with the system, but not all. See the [Supported Hardware List](#) for an up-to-date list of supported antennas.