

# Touchscreen Main Menu

help.grandcare.com → help for caregivers → touchscreen main menu

## Overview

You can decide which buttons appear on the touchscreen of your loved one's system, and you can set the order in which they appear. These settings can be changed by users with sufficient security permissions.

## Select Main Menu Buttons

- Log in to the Online Care Portal
- Open the Care Menu
- Click the "Touchscreen" button
- The section called "Touchscreen Apps" lists all the possible buttons for the Touchscreen. Each provides access to an application for the touchscreen. Any item that is checked will appear as a button on the Touchscreen. If you uncheck an item, that button will no longer appear on the Touchscreen.
- Make your selections
- Click the "Save Features" button

If you select more buttons than will fit on a single page of the Touchscreen menu, then a second menu screen will be added, and arrows will appear on the menu for navigating between pages on the Touchscreen menu.

**GrandCare SYSTEMS Touchscreen Setup** [Main Settings](#) [Options](#) [Help](#)

[Show Touchscreen](#)

### Slideshow Features

- Headline News
- Weather Forecast
- Word Definitions
- Nostalgia
- Trivia
- Quotations
- Spiritual Offerings
- Stock Pictures
- Daily Events
- Monthly Calendar

[Save Features](#)

### Touchscreen Apps

- Pictures
- News
- Letters
- Messages
- Games
- Brain Exercises
- Music Programs
- Calendar
- Accessories
- Medication
- Websites
- Wellness
- People Folders
- System
- Caregiving
- CHF Assessment
- Caller ID
- Music
- Videos
- Smart Home

[Save Features](#)

## Button Order

You can rearrange the buttons on the Main Menu screen.

- Click on the "Options" button on the Touchscreen setup window
- Click the "Manage Apps" button

This displays a list of all the Main Menu buttons. For each button, there are 3 useful settings shown.

- If the button is checked, it is selected to appear as a button on the Main Menu, and you can check or uncheck it here
- The priority number controls where the buttons appear on the menu - buttons with low numbers appear before buttons with high numbers
- You can set items to be protected by a 4-digit PIN number, for privacy or security

When you are finished with the settings, scroll to the bottom of the screen and click the "OK" button.

Application	Show	Priority	Password
<b>Pictures</b> <small>Show pictures</small>	<input checked="" type="checkbox"/>	1	<input type="checkbox"/>
<b>News</b> <small>Show news and weather</small>	<input checked="" type="checkbox"/>	2	<input type="checkbox"/>
<b>Letters</b> <small>Show email</small>	<input checked="" type="checkbox"/>	3	<input type="checkbox"/>
<b>Messages</b> <small>Show messages</small>	<input checked="" type="checkbox"/>	4	<input type="checkbox"/>
<b>Games</b> <small>Show games</small>	<input checked="" type="checkbox"/>	5	<input type="checkbox"/>
<b>Brain Exercises</b> <small>Show brain exercises</small>	<input checked="" type="checkbox"/>	6	<input type="checkbox"/>
<b>Music Programs</b> <small>Music Therapy</small>	<input checked="" type="checkbox"/>	6	<input type="checkbox"/>
<b>Calendar</b> <small>Show calendar events</small>	<input checked="" type="checkbox"/>	7	<input type="checkbox"/>
<b>Accessories</b> <small>Tools, apps, and more!</small>	<input checked="" type="checkbox"/>	8	<input type="checkbox"/>

## Show Touchscreen

You can see the changes you made to the Touchscreen. Click the "Show Touchscreen" button at the top of the Touchscreen module window. It shows you a reproduction of your Loved One's system, and even lets you operate many of the features. You will not be able to see what your Loved One is doing on the system, nor will your Loved One be able to see your screen actions. You can both be using the system at the same time without interference.