

Add Inovonics Devices

help.grandcare.com → technical help → add inovonics devices

Adding a Device

In order for the device to send readings to the GrandCare system, it must be configured on the system.

- Log in to GCManage
- Open the Care Menu
- Open the appropriate module
 - For motion, door, and bed sensors, open the Activity module
 - For action buttons, open the Notifications module
 - For medication dispensers, open the Medication module
- Click the "Add New Device" button
- Enter a Location and Description for the device
- Select the proper device from the drop-down selection list
- Click the "Next" button
- Enter the device Serial Number (found on a sticker on the bottom of the device) - Enter the complete number, including any tiny numbers preceding the full-size numbers
- Click the "Finish" button
- Note: Some motion sensors have a setting jumper in them for FIX/VAR mode. If that's the case, set the jumper to FIX. This sets a fixed 180-second check-in time with the antenna. This mode works the best with CoMo software.

Additional Information

- How to add and configure the Inovonics Antenna
- How to use Motion Sensors, add rules, view readings, etc.
- How to use Door Sensors, add rules, view readings, etc.
- How to use Bed and Chair Sensors, add rules, view readings, etc.
- How to use Action Buttons, add rules, view button history, etc.