

Wellness Devices


Wellness Devices, such as a weight scale, pulse oximeter, glucometer, and/or blood pressure monitor, can be added to the GrandCare System to record health information. These Wellness Devices are battery-operated and Bluetooth-enabled to communicate wirelessly with the system.

Caregivers can log onto the [Online Care Portal](#) to view health readings and to setup custom [Wellness Notifications and Acknowledgements](#).

[Viewing and Printing Health Reports](#)

[Wellness Notifications and Acknowledgements](#)

[Battery Report](#)

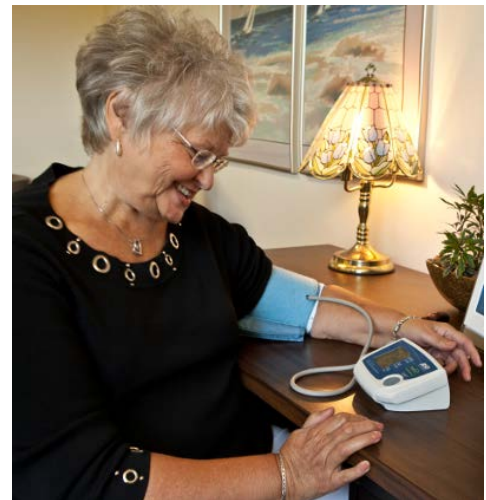

Wireless Bluetooth

All wellness devices communicate with the system via Bluetooth. **Systems that include health devices must have a [Bluetooth Antenna](#).**



Taking Wellness Readings

Follow all manufacturer-provided directions for setting up and using wellness devices, including battery installation, taking accurate readings, and device care.

All wellness devices within range of the system, **except for the glucometer**, will automatically upload readings to the GrandCare System.

Specific Wellness Devices

There are variety of device models that can be used with the GrandCare System. For a full listing of supported health devices, see our [Supported Hardware List](#). Use the manufacturer's directions that came with the device to identify the model you have.

Blood Pressure Monitor

Taking and Uploading a Blood Pressure Reading

The blood pressure monitor comes with manufacturer-provided directions for taking a safe and accurate reading.

Once the reading has been taken, it will automatically be sent to the system.

- [Blood Pressure Monitor: Additional Information and Instructions](#)

Weight Scale

Taking and Uploading a Weight Reading

The weight scale comes with manufacturer-provided directions for taking a safe and accurate reading.

Once the reading has been taken, it will automatically be sent to the system.

- [Weight Scale: Additional Information and Instructions](#)

Pulse Oximeter

Taking and Uploading an O₂ saturation Reading

The pulse oximeter comes with manufacturer-provided directions for taking a safe and accurate reading.

Once the reading has been taken, it will automatically be sent to the system.

- [Pulse Oximeter: Additional Information and Instructions](#)

Glucometer

Taking and Uploading a Glucometer Reading

The glucometer comes with manufacturer-provided directions for taking a safe and accurate reading.

When the blood glucose reading appears on the meter's display screen, press the Down Arrow button once to send the reading to the system.

Uploading Multiple Readings

If the resident has been away from home - on vacation, for example - and has taken numerous glucometer readings while away, you can upload all the readings at once by following these steps:

1. Make sure the glucometer is off (in a powered-down state).
2. While the glucometer is off, press the Down Arrow button once. This will upload all the readings currently stored on the glucometer.

- [Glucometer: Additional Information and Instructions](#)