Inovonics Antenna Configuration

help.grandcare.com \rightarrow technical help \rightarrow inovonics antenna configuration

How to Install an Inovonics Antenna

- When you first use the Inovonics antenna, open the case for it, and attach the ACC643 cable
- Plug the antenna into an electrical outlet that is *not* switch-operated
- The Inovonics antenna is a serial device, so plug it into a serial port on the system if there is one, or use a Serial-

USB adapter to plug the antenna into an available USB port

- At the system, press the System (Gear) button
- Enter the 4-digit System PIN (which is set, and can be viewed, on the Advanced Settings screen in the Care Menu)
- Press "Peripherals"
- On the Peripheral page, the Inovonics antenna is referred to as the "EN4000"
- In the EN4000 row, choose the port
 your choices will be specific to the system hardware
- Press "Turn On"
- Restart peripherals when prompted
- If installed correctly, the row will now be highlighted in green. If there is a problem with the installation, the row will be highlighted in red.
- After it is installed correctly, you can begin to add Inovonics devices, such as motion sensors, door sensors, etc.

1320		Peripherals		
82°F		Enabled	Port	Debug
Main Menu	Phone Line Interface			Log
Slide Show	W800	Turn On	USB 4-2	* Log
	Bluetooth			* Log
	AeonLabs Z-Stick	• Turn On	USB 4-2	* Log
	EN4000	Turn On	USB 4-2	* Log
	Restart Drivers		 GCSystem Log 	⋆ GCMain Log
Groud Caro	Pack			Home

